Product Name: MANI® OPHTHALMIC KNIFE

Do not reuse 1 Box / 6 Pieces, 1 Bag / 1-100 Piece (s)

English

Warnings 1) The user must be fully acquainted with the surgical procedures and techniques.

Contraindications and Prohibitions 1) Do not use on any patients who are sensitive or allergic to metals.

[Precautions when using Marking Straight Knife] Marking Straight Knife is a Straight Knife that has two lines of 1.0 mm and 1.2 mm in length marked on the blade.

[Precautions when using Short-Head Silt Knife] Short-Head Silt Knife is a Silt Knife that has lines to be used as guides for creating a tunnel.

[Precautions when using Double-Step Knife] Double-Step Knife has a reference line on the blade which can be used as a guide to prevent inserting the knife deeper than intended.

[Shape and Structure] 1. The product is a sterile knife with handle. Two types are available. One type comes stored inside a case and the other type is a safety type with the blade protected by a cover.

[Performance, Purpose and Effect] This knife is to be used to incise and penetrate into eyeballs.

[Specification of the Item] Vickers hardness is 500HV or more for the blade made of stainless steel.

[Operation and Use] 1. Use in accordance with ordinary incision procedures. 2. Handling of the case 1) Taking out the knife With special care to keep the blade off any other part of the case...

[Precautions for Use] 1. Precautions for use This knife might cause sensitization and allergy to metals. 2. Important basic precautions 1) Use the knife as soon as it is unpacked.

[Precautions for use of the arched blade] 1. Location of the wound The location of creating the incision is different from that of normal flat blade use because the arched blade is a three dimensional blade.

Table with 3 columns: Arched blade, Height, Rough standard of location to create incision. Rows include 1.8mm, 2.0mm, 2.8mm, 2.9mm, 3.0mm, 3.2mm, 3.4mm.

Marking of the planned area of the incision would give a guide for incision. In sclera cornea incision, the iris can be incorporated in the incision wound.

[Manufacturer] MANI, INC. 8-3, Kiyohara Industrial Park, Utsunomiya, Tochigi 321-3231 Japan

[EU Authorized Representative] Obelis s.a. Bd. Général Wahis 53, 1030 Brussels, Belgium

recommended. 4. Others 1) Use an IOL injector for Intraocular Lens Implantation to avoid any damage to the incision wound.

[Precautions when using Marking Straight Knife] Marking Straight Knife is a Straight Knife that has two lines of 1.0 mm and 1.2 mm in length marked on the blade.

[Precautions when using Short-Head Silt Knife] Short-Head Silt Knife is a Silt Knife that has lines to be used as guides for creating a tunnel.

[Precautions when using Double-Step Knife] Double-Step Knife has a reference line on the blade which can be used as a guide to prevent inserting the knife deeper than intended.

[Storage and Expiration Date] 1. Storage Avoid exposure to high temperatures and humidity, direct sunlight and water.

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